PL

**CALCIPAST FORTE**

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EN

**CALCIPAST FORTE**

Calcium hydroxide paste with iodoform and chlorophenol

**BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY**  
PRODUCT FOR DENTAL USE ONLY

**COMPOSITION:** calcium hydroxide, iodoform, chlorophenol, camphor, barium sulphate, excipients.

**INTENDED USE**

CALCIPAST FORTE is intended for use in dental treatment for temporary root canal filling.

It supports the reconstruction of damaged periapical tissues. It dries root canal exudates.

It has re-mineralizing effect on dental tissues thanks to its high content of calcium hydroxide. It supports restoration of damaged periapical tissues. pH=8. Radiopaque.

**INSTRUCTIONS FOR USE**

Immerse the supplied applicator in ethanol before use, then install the applicator on the product syringe. Measure the canal length, adjust the endostop on the applicator to the length 2-3 mm shorter than the measured canal length. Apply the paste into the canal, remove excess paste and close the cavity securely. Leave the filling in the root canal for 1 week up to 30 days. A reusable product with a single-use applicator. Re-using the applicator may cause the risk of reinfection.

**CONTRAINDICATIONS**

Do not use for patients who are hypersensitive to the product components.

**PRECAUTIONS**

Harmful if swallowed. Irritating to the mucous membranes of the oral cavity and skin. May cause serious eye damage.

In case of contact, rinse immediately with plenty of water and seek medical advice. If swallowed, do not induce vomiting.

Drink plenty of water. Seek medical advice. Use a rubber-dam when working with the product. Use protective gloves, glasses and protective clothing.

**STORAGE**

Keep in original packaging below 25 °C. Keep out of reach of children. Expiry date is placed on the main container. Once container is open, the expiry date does not change if the container is securely closed after each use. Protect from sunlight.

**HANDLING THE PRODUCT CONTAINER**

Empty containers should be disposed of responsibly or returned to the manufacturer.

**PACKAGING**

Syringe containing 2.1 g of the preparation, disposable applicators.

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It has re-mineralizing effect on dental tissues thanks to its high content of calcium hydroxide. It supports restoration of damaged periapical tissues. pH=8. Medio de contraste en imágenes radiográficas

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